T2m 50 RM

Réarmement manuel
Manual reset

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AUTOMATIC CHANGE-OVER UNIT WITH MANUAL RESET

It enables an uninterrupted gas supply from 2 groups of cylinders or bundles to a medical equipment or a medical pipeline system.

The **CHANGE-OVER** to the secondary side is **AUTOMATIC** when the primary side is depleted.

The **ONLY MANUAL OPERATION** is to reset the unit by **MOVING THE LEVER** when the empty cylinders are replaced.

WORTH NOTING: Preventive maintenance may be carried out on site without interrupting the gas supply.

Specially developed for the handling of all medical gases, oxygen included.

The T2m 50 RM unit is available in 2x1 to 2xn cylinders or bundles version.

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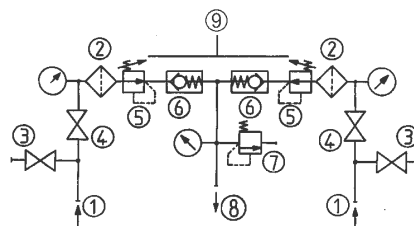
* Outlet regulator BP 300: preset static outlet pressure 9 bar

STANDARD ARRANGEMENT

- Inlets: Ø 21,7 x 1,814 SI male
- Filters
- HP purge valves (collectable: brass nipple for OD 10 mm welded tube)
- Shut-off valves
- Pressure regulators
- Non return valves
- Relief valve (collectable: brass nipple for OD 10 mm welded tube)
- Outlet: double ferrule compression fitting for OD 12 mm tube
- Reset lever

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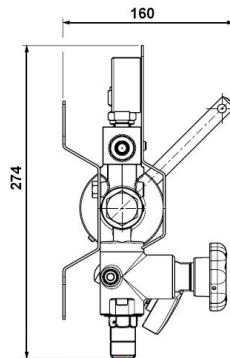
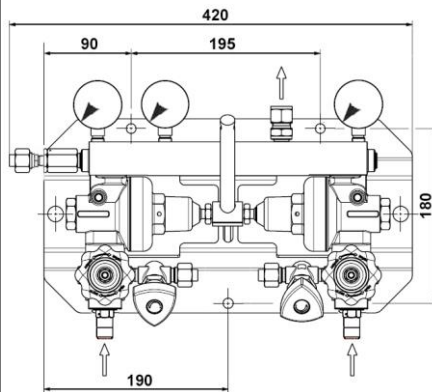
MATERIAL

Body: brass
Holder: metallic panel
Diaphragm: NBR
Valve seat and seals: according to gas

ACCESSORIES

- Flexible hoses
- Joining pipes
- HP pigtails
- Cylinder holders
- Carry-on manifolds or wall mounted collectors

OPTIONS

- Outlet shut-off valve
- Second stage outlet pressure regulator
- Emergency and maintenance inlet (E.U.M.)
- Pressure switch for change-over signal (BP)
- HP pressure switch

MAINTENANCE

A maintenance kit is available and duly referenced in the data sheet delivered with the equipment.

Maintenance may be carried out either on site by trained personnel, in our plant or at your local authorized service shop.

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Exemple : T2_m 50 RM - O₂ - 2x2 bouteilles - E.U.M

ORDER: please mention

- Detailed reference
- Gas type or gas mixture components
- Number of cylinders or bundles
- Selected options and accessories

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All views, descriptions and features are given only for information. CAHOUE reserves the right to modify its products without notice.

52, rue de Lagny
93100 MONTREUIL - FRANCE
Tél : +33 1.41.72.90.00
Fax : +33 1.41.72.90.01
E-mail : cahouet@cahouet.com
www.cahouet.com